

# “THE HEART AND SOUL OF INFECTION CONTROL”™

## Enviropouch Instructions for Use (IFU)



1) Clean all instruments thoroughly by soaking in a neutral pH detergent solution, hand scrubbing, or clean in an ultrasonic cleaner.

2) Place cleaned instruments & cassettes in Enviropouch, invert the cuff or fold the inseam, and seal cuff or flap, respectively, with steam process indicator tape in vertical direction.

3) Always insert an indicator to show exposure to the sterilization process.

4) Sterilize in a steam autoclave at 250 F 16 to 18 psi for 20 minutes or 270 F 30 psi for 5 minutes. Follow steam autoclave manufacturer directions.

5) Remove the Enviropouch immediately to air dry or machine dry on low heat.

6) Remove the autoclave indicator tape in downward direction, from the top of the vertically sealed cuff or flap.

Enviropouch is **laundry-free**, reuseable in the steam autoclave process with **no laundering necessary**.

Enviropouch has useful life of 200 sterilization cycles.

Storage conditions require normal ambient conditions.

P.O. Box 2295  
308 D-Bob  
Industrial Drive Suite 3  
Peachtree City, GA 30269

Phone: (877) 770.632.1991  
Fax: 770.742.0224



Check us out!  
[WWW.ENVIROPOUCH.COM](http://WWW.ENVIROPOUCH.COM)